



ARRO System *Sample Kit*



ARRO System Sample Kit Contents

- 2x electronic measurements module
- Ø1.5 x 40mm Calibrated Drill Bit
- Ø1.5mm Soft Tissue Guide
- Cordless Orthopaedic Drill
- Radius sawbone
- Manual depth gauge
- Charging Cable x2 & Charger

Please note the following before use:

When the drill bit is inserted in the drill, **do not** stand the drill upright. Instead place it flat on a surface.



ARRO System Instructions For Use

The sawbone has been secured within the kit so it can remain in place during product demonstrations.



Insert the module onto the guide, push until you hear a click.



Attach the drill bit to the drill, then insert the drill bit into the guide.



Place the guide in contact with the sawbone. Whilst in contact, press one side of the module to zero it out.
Drill as per your usual practice.

Recharging the modules

When recharging the modules, ensure they are disconnected from the soft tissue guide first.



Ensure the module is disconnected from the guide. Plug the USB-C cable into the module, connect the other end of the cable into the charger.



Red LED light:
Module is charging



Red & green
LED light:
Module is charged